

# Comprinet® pro

Professional care with unique comfort



The modern  
anti-embolism  
stocking for the  
prophylaxis of deep  
vein thrombosis

Product information




# Comprinet® pro

Professional care with unique comfort

By combining medical efficacy with safety, comfort and convenience for patients and users, Comprinet® pro anti-embolism stockings effectively help prevent deep vein thrombosis and reduce the risk of pulmonary embolism in immobile patients.

- The special knitting technique ensures a highly **two-way stretchable fabric**. The **anatomical shape** combined with this stretchability ensures Comprinet® pro fits on a wide range of leg shapes and sizes, while maintaining pressure profile.
- With **circumference measurements** up to 53 cm at the calf (c-level) and 86 cm at the thigh (g-level) for sizes Small, Medium and Large, and 90 cm for Extra-large, Comprinet® pro is suitable for the majority of patients.
- The soft **skin-friendly** and **air-permeable fabric** provides unique wearing comfort.
- The **skin-friendly and robust thigh band** reduces the risk of allergic reactions and does not fold back or wrinkle.
- The **reciprocated heel** allows for proper positioning of the stocking.
- The **inspection window** set back from toe extremity on the dorsal side of the foot allows easy and rapid safety assessment.
- The **colour combination on the inspection window** provides an easy system for quick size identification and facilitate sorting.
- The **high durability** of the material makes Comprinet® pro therapeutically reliable.

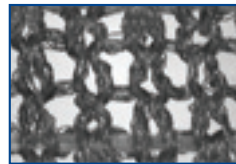

Highly stretchable and air-permeable fabric




Skin-friendly thigh band

Reciprocated heel

Colour-coded inspection window

## Decreasing pressure gradient

Comprinet® pro applies a continuously decreasing pressure gradient from distal to proximal. Comprinet® pro effectively helps prevent deep vein thrombosis by increasing blood velocity and helping reduce venous stasis.

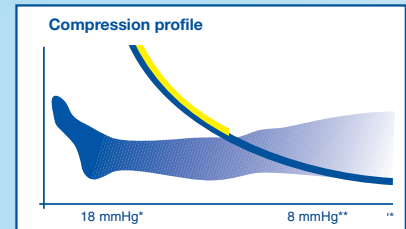

\* Nominal compression for average ankle size  
\*\* Nominal compression for average thigh size

## Indication

Thrombosis prophylaxis in surgical patients and patients requiring prolonged periods of immobilisation.

## Contra-indications

Ischemia (e.g. advanced arterial disease), uncontrolled congestive heart failure, untreated septic phlebitis, phlegmasia coerulea dolens.

## Cautions

Skin infections, weeping dermatoses, incompatibility to fabric of garment, impaired sensitivity of the limb. Use in patients receiving vasopressor drugs that can cause peripheral vasoconstriction. Altered mental status.

## Other recommendations

Incorrectly measured and badly fitting stockings may cause adverse events. Open leg wounds must be covered with an appropriate dressing. Perform periodic neurovascular assessments using inspection window. Remove stockings at least daily for skin hygiene and inspection.

## Duration of application

Days up to several weeks.

## Comprinet® pro thigh high

Size	Width in cm		Length in cm		Colour code	Item No.
1	small	23-30	regular	64-80		46335
2	small	23-30	long	80-97		46336
3	medium	30-38	regular	64-80		46337
4	medium	30-38	long	80-97		46338
5	large	38-46	regular	64-80		46388
6	large	38-46	long	80-97		46389
<b>NEW!</b> 7	x-large	46-53	regular	64-80		77125
<b>NEW!</b> 8	x-large	46-53	long	80-97		77128

## Comprinet® pro knee high

Size	Width in cm		Colour code	Item No.
1	small	23-30		77026
2	medium	30-38		77027
3	large	38-46		77028
<b>NEW!</b> 4	x-large	46-53		77124

## Composition thigh high

74% Polyamide  
20% Elastane  
6% Neoprene

## Composition knee high

74% Polyamide  
26% Elastane

