



Dynacast® Prelude Solo

Unpadded Precut Splinting System



**New (ILS)
Interlocking
Strength
Technology**

- Excellent for Trauma and Post-Op applications
- No mess
- Easy application
- Clean
- New interlock knit technology
- Produces strong and durable splint
- Thin profile
- Single unit application



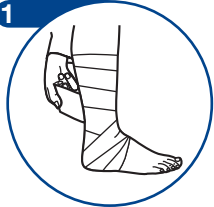
Dynacast® Prelude Solo

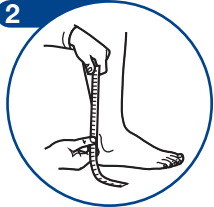
Unpadded Precut Splinting System

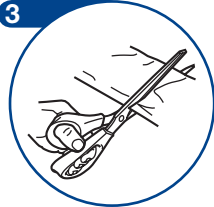
An alternative to traditional plaster of Paris slabbing.

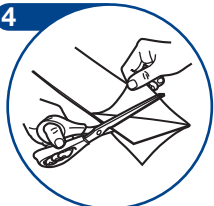
The Dynacast® Prelude Solo Unpadded Precut is a fibreglass splinting system that comes in a variety of widths and lengths, and is stronger and more versatile than traditional plaster of Paris splints or slabs. Dynacast® Prelude Solo splints are easy to apply and clean, and give strong support to soft tissue injuries and stable fractures.


Preparation Guidelines*


- 


1 Pad limb before opening splint packaging.
- 


2 Measure the injured extremity to determine the size of the product to use.
- 


3 Cut splint to desired length if required and remove from foil pouch.
- 

4 Do any custom cuts in material before applying water.
- 

5 Wet one side of splint with water bottle or the tap using luke warm water (23-30°C).
- 

6 Fold and squeeze to remove excess water.
- 

7 Stretch splint back to original shape and smooth to remove creases.
- 

8 Apply splint over the padded limb.
- 

9 Wrap with bandage. Mould splint to patient's limb for 2-4 minutes until splint sets. Monitor patient according to standard procedures.

- **Stronger than traditional plaster of Paris slabs**
 - reduces weight and bulk to patient
- **Cleaner than traditional plaster of Paris slabs**
 - no dust and mess
- **Unique liner enables gloveless application**
- **Thin profile**
 - excellent mouldability
- **ILS (interlocking strength) technology**
 - strength and durability
 - minimises wrinkling

Ordering Information

Size	Order Quantity	Description	Order Code
5cm x 25cm	Box of 10	Dynacast® Prelude Solo Unpadded Precuts	72963-00
7cm x 30.5cm	Box of 10	Dynacast® Prelude Solo Unpadded Precuts	72963-01
7cm x 89cm	Box of 10	Dynacast® Prelude Solo Unpadded Precuts	72963-02
10cm x 38cm	Box of 10	Dynacast® Prelude Solo Unpadded Precuts	72963-03
10cm x 76cm	Box of 10	Dynacast® Prelude Solo Unpadded Precuts	72963-04
13cm x 76cm	Box of 10	Dynacast® Prelude Solo Unpadded Precuts	72963-05
15cm x 30.5cm	Box of 10	Dynacast® Prelude Solo Unpadded Precuts	72963-06



* Splint may be used with or without liner.



BSN medical, Ltd. P.O. Box 258, Willerby, Hull HU10 6WT
 Tel: 01482 670100 Fax: 01482 670111 www.bsnmedical.com
 Email: advice.uk@bsnmedical.com

US and Foreign Patents Pending
 VR/PRELSOLOSS/00180/1105
 ® = Registered Trademark
 © BSN medical, Ltd. 2005