

Comprilan®

Short Stretch Compression Bandage



Indications of use:

- Venous ulcers
- Venous insufficiency
- Edema
- Lymphedema
- Thrombophlebitis
- Post sclerotherapy

Unique features:

- 100% cotton bandage
- Easy-care and durable
- High working pressure and low resting pressure
- Anatomically conforming
- Interlocking weave

Comprilan®

Short Stretch Compression Bandage



Comprilan® is a short stretch compression bandage made from 100% cotton and is specifically designed for the management of venous leg ulcers, lymphedema and edema.

Unlike long stretch bandages, Comprilan® provides both compression and high resistance to stretch to increase venous and lymphatic return.

Features & Benefits:

- Interlocking weave provides short stretch to ensure consistent compression
- Anatomically conforming allowing freedom of movement
- Provides high working pressure and low resting pressure
- Easy-care and durable, Comprilan® can be washed and reused
- 100% cotton bandage, cool and light, the open weave fabric allows the skin to breathe

Wrapping Technique

Simple spiral bandaging technique is ideal when managing venous leg ulcers.

1. Begin bandaging from the medial aspect at the Basal joint of the toes towards the lateral aspect of the foot.
2. Repeat step 1 to secure bandage.
3. The bandage continues with a turn across the metatarsals.
4. Take the bandage over the lower heel and return over the metatarsals.
5. Take the next turn over the upper heel and Achilles Tendon continuing medially.
6. Cover the ankle by wrapping the bandage spirally.
7. Continue spiralling the bandage up the leg, overlapping each turn by half of the bandage width.
8. Finish bandaging approximately two fingers width below the Tibial tuberosity.
9. If a second bandage is applied, it can start at the Basal joints or the ankle but should be spiraled in the opposite direction.

Ordering Information

Order Number	Size	Box Content	Case Content
77187	4 cm x 5 m (1.6" x 5.5 yds)	1 roll	24 boxes
01026000	6 cm x 5 m (2.4" x 5.5 yds)	1 roll	20 boxes
01027000	8 cm x 5 m (3.1" x 5.5 yds)	1 roll	20 boxes
01028000	10 cm x 5 m (3.9" x 5.5 yds)	1 roll	20 boxes
01029000	12 cm x 5 m (4.7" x 5.5 yds)	1 roll	20 boxes
77188	10 cm x 10 m (3.9" x 10.9 yds)	1 roll	20 boxes
77189	12 cm x 10 m (4.7" x 10.9 yds)	1 roll	20 boxes

CAUTION: Fixation clip contains natural rubber latex which may cause allergic reactions.

NOTE: Please refer to the product label and/or package insert for full instructions on safe use of this product.

Fiber Content: 100% cotton bandage

Related Lymphedema Management Products:

- Tricofix®
- JOBST® Ready-to-Wear Armsleeve
- Artiflex®
- JOBST® Ready-to-Wear Gauntlet
- Elastomull®
- JOBST® Foam
- Elvarex®



Simple Spiral



To learn more about JOBST® products and services, call 1-877-978-5526 or visit www.jobst.com

JOBST® - a brand of **BSN** medical



Comfort, Health and Style!



BSN Medical Inc.
4455, autoroute Laval ouest, suite 255
Laval, (QC) Canada H7P 4W6

® Trademark of BSN Medical
© 2007 BSN Medical